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**Instruction Manual  
Nursing Bed  
INOVIA II /  
INOVIA II 100**

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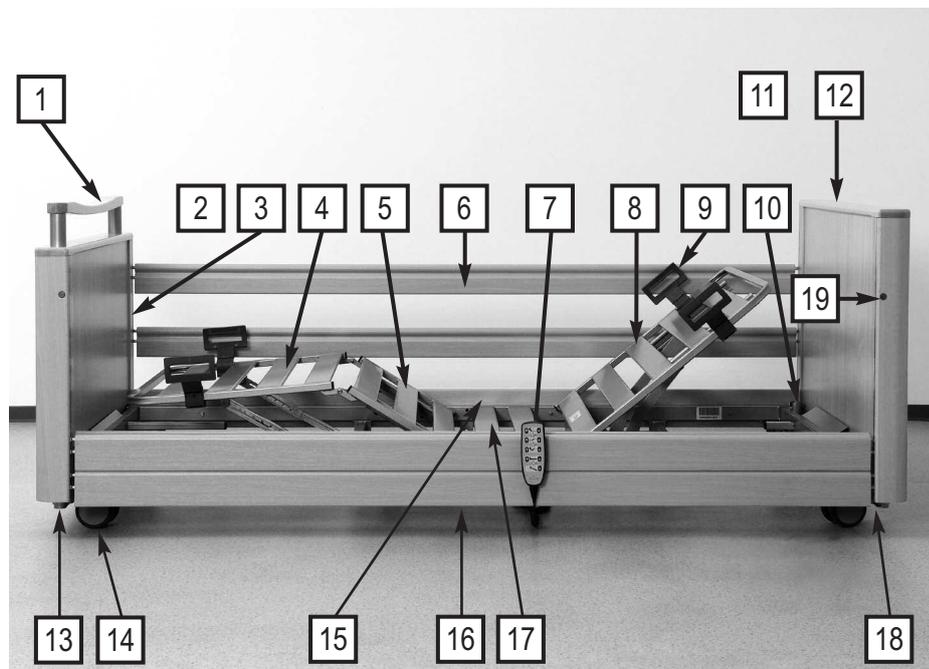
### Please notice:

In the text of these instructions square brackets and bold figures = [00] refer to the parts of the nursing bed.

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## PRODUCT DESCRIPTION

### NORMAL OPERATING PROCEDURE

- The bed INOVIA II / INOVIA II 100, hereafter called bed, serves as a comfortable solution for positioning and easing the care of infirm, frail people in senior or nursing homes. Additionally it was developed as a comfortable solution for in-home care of infirm, handicapped or frail people. It will support this care.
- This bed can be used for care under the instructions of a physician and can serve in diagnosing, treatment and observation of a user. It is therefore equipped with a locking mechanism for the electrical adjustment functions.
- This bed has no special connector possibilities for a potential equalization. Consider this before combining with additional electrically operated devices.

Further references about additionally applicable precautions can be found:

► In the instructions for the additional electrically operated devices (e.g. compressors for positioning systems, infusion pumps, nutrition probe ...)

■ The maximum long-term load for this bed is 225 kg (user and accessories). Under this load, the bed can be operated without restriction.

■ The permitted weight of the user depends on the total weight of the accessories attached at the same time (e.g. respirators, infusions,...)

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### Example:

Weight of accessories (with mattress):	Max. permitted weight of user
10 kg	215 kg
40 kg	185 kg

▶ Observe, especially with users in poor health condition, the safety tips in chapter 4.

■ This bed is useful for multiple re-use. Bear in mind the necessary requirements:

■ While the user also is lying in bed, it is allowed to drive the bed within the building.

**This bed may only be operated under the operating instructions laid out in this instruction manual. This bed must not be altered in any way. Any other use is deemed to be inappropriate.**

### CONTRAINDICATIONS

■ In case of using safety sides this bed is only suitable for users who do not fall below the following minimum body size/weight:

- ▶ Height: 146 cm
- ▶ Weight: 40 kg
- ▶ Body Mass Index „BMI“: 17

Calculation of BMI =

Weight of user (kg) / Height of user (m)<sup>2</sup>

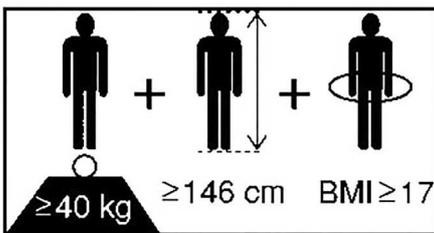
**Example a**

41 kg / (1.5 m x 1.5 m) = 18.2 = **OK!**

**Example b**

35 kg / (1.5 m x 1.5 m) = 15.6 = **Not ok!**

■ Owing to the smaller limbs of users with lower body measurements/weight, there is an increased risk of entrapment between the open spaces of the safety sides when safety side systems are used.

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## STRUCTURAL DESIGN

The bed is shipped disassembled. It can be transported even in apartment homes. It consists of two end boards (head and foot boards) which are each made up of a tubular frame encased with wooden boards; an encased reclining surface frame and four side rail beams. The bed rests on steerable castors with individual axle braking.

### Reclining surface frame

The reclining surface is divided into four sections: a removable back rest, a fixed middle section and a movable thigh rest and lower leg rest. The back- and thigh rest can be adjusted with the aid of electric motors. The horizontal height of the reclining surface can be adjusted and inclined to a low foot end position. All adjustment functions are controlled by a hand switch.

### Side Rails

The bed can be fitted with a variety of different safety sides.

### Electric adjustment system

The electrical adjustment system of this bed is error protected, flame-retardant and consists of:

- the central drive unit with the drive motors for the back and thigh rest. An external transformer creates a low voltage, which is safe for user. All drive motors and the hand switch are hooked up to the central drive unit by plug-in connections and work with the safety low-volt-age;
- two drive motors for the height adjustment of the reclining surface;
- a lockable hand switch with a strong hook.

### DESCRIPTION OF MATERIALS

The bed is mostly constructed out of steel sections, their surfaces have been polyester-powder-coated, or with a metallic coating of zinc and chrome. The end boards, the side rail beams and the spring wood slats consist of wood and derived timber products whose surface has been sealed.

**All surfaces are harmless to skin contact.**

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The assembly is taken care of at the destination by qualified personnel supplied by the operator.

The bed ships in 5 cartons:

Carton 1: Reclining surface frame including drive motors (pre-assembled)

Carton 2: Tubular frame head section with drive motor

Carton 3: Tubular frame foot section with drive motor

Carton 4: 4 side rail beams; 2 side boards

Carton 5: Wooden boards, head and foot section

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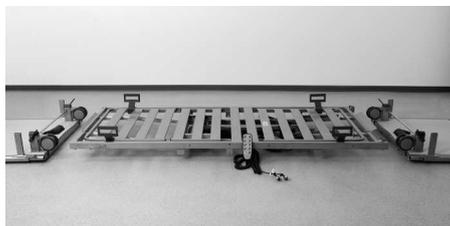
## ASSEMBLY

### PREPARATION

☞ Position the reclining surface frame [17] flat on the floor. Top side up.

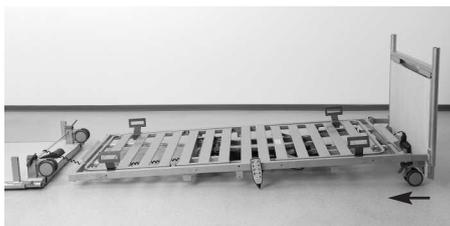
☞ Position the tubular head section frames [2] + [11] flat on the floor. The castors point upward.

☞ Connect both drive motors of the end boards to the drive unit.

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### ASSEMBLY OF THE RECLINING SURFACE FRAME

Start the assembly at the head section. Join the tubular frame/head section [11] with the reclining surface frame [17] together.

☞ Lift the reclining surface frame at the head end and push the two support brackets from the tubular frame/head section [11] into the tubes of the reclining surface frame [17] to the limit.

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☞ Screw the reclining surface frame firmly to the support brackets.

For this use two Allen screws M 8 x 20 mm on each side.

☞ Tighten the 4 Allen screws firmly with a size 5 Allen wrench.

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☞ Repeat the process with the tubular frame/foot section [2].

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### ASSEMBLY OF THE SIDE BOARDS

☞ Raise the reclining surface to the middle horizontal position.

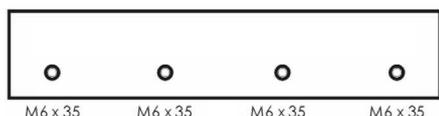
☞ Raise the back and thigh rests [8] + [5] half way. Lift the lower leg rest [5] to the horizontal position.

**Notice:** The two side boards are identical.

The castors must be locked.

☞ Screw the side boards [15] to both sides of the reclining surface frame with 4 Phillips head screws respectively; fit the middlest screw first.

Take care, that the low side of the side board point upward.

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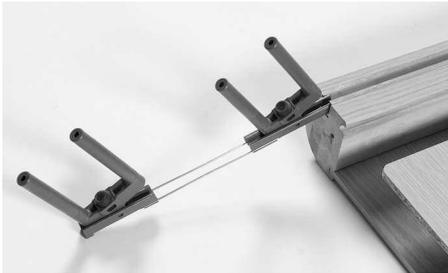
## ASSEMBLING THE SIDE RAIL GUIDE

On each corner of the wooden boards [5] + [14] you will find a guide bar embedded in the wooden boards for the side rail beams [6].

When inserting the side rail guide, please adhere to the following procedure:

☞ Insert the plastic slide from below into the guide bar. The pointed end of the plastic slide must face upward.

☞ Insert the sliding plastic retainers for the safety sides right into the guide bars on the head and footboards.

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## ASSEMBLY OF THE WOODEN BOARDS

☞ Finally put the wooden board/head section [12] and the wooden board/foot section [3] respectively, on the tubular frame head and foot section [11] + [2].

Insert the wooden boards all the way.

☞ Tightly screw the lower section of the two wooden boards [3] + [12] on both sides to the tubular frame [2] + [11] with the M 6 x 30 mm [a] Philips head screws.

☞ Screw on the strip handle on the foot board using screw [b].

These steps of procedure have to be made only for the initial assembly. For further assemblies/disassemblies the wooden boards should stay on the tubular frame.

## ASSEMBLY OF THE SIDE RAIL BEAMS

On all four corners of the bed you will find a guide bar embedded in the wooden boards [3] + [12] for the side rail beams [6].

In each of these runs a plastic slide with 4 plastic plugs. The plastic plugs fit into the slots on the ends of the 4 side rail beams.

The side rail beams can be raised or lowered, as needed.

Start with the end board/head section:

☞ Fit the side rail beams onto the plastic plugs.

The top and bottom of the side rail beams are flattened and rounded respectively. The round side of the bars must point upward.

Now go to the end board/foot section.

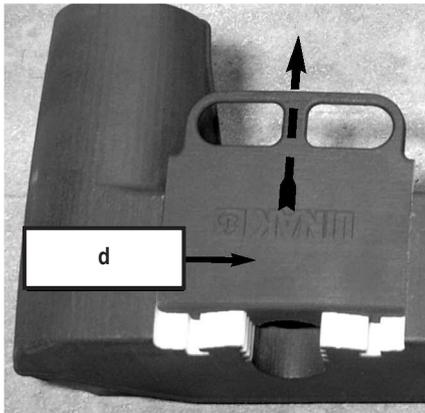
☞ Loosen the Allen screws on the reclining surface frame.

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- ☞ Pull the end board and the reclining surface frame apart.
- ☞ Fit the side rail beams onto the plastic plugs.
- ☞ Mount the two other side rail beams on the opposite side of the bed the same way.
- ☞ Push the end board and the reclining surface frame together.
- ☞ Tighten the 4 Allen screws firmly with a size 5 Allen wrench.
- ☞ Screw the reclining surface frame to the support plates.
- ☞ Push the side rail beams upwards until they click into place.

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

### ASSEMBLY OF THE DRIVE UNIT (PRE-ASSEMBLED)

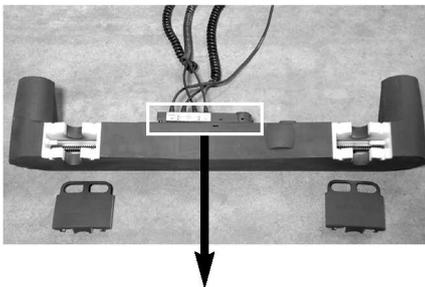- ☞ Unpack the drive unit [16] and all connecting cables.
- ☞ Place the drive unit on the floor with the two cover slides [d] uppermost.
- ☞ Using the finger holes, pull the two cover slides [d] off laterally and lay them aside.

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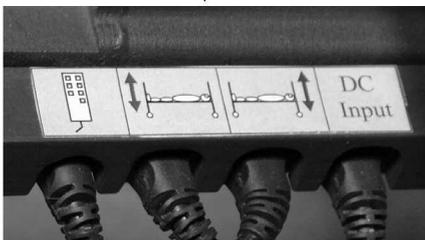- ☞ Insert the 4 plugs of the respective cables, 2 spiral, the 24-V-connector cable and the hand switch cable [7] into the marked sockets of the drive unit [16].

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- The plugs are polarised.
- The two spiral cables for the drive motors are identical.
- Push the plugs firmly into the sockets to seal against moisture.

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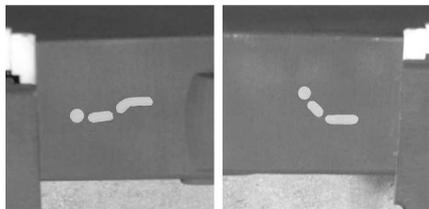
☞ Place the reclining surface frame upright with head part down in order to work on the underside.

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Warning! Secure the frame in this position.

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
☞ Retrieve the two cover slides and to pocket them.

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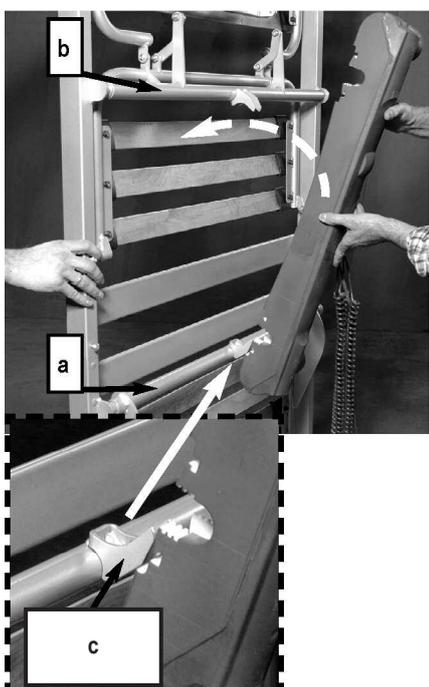The drive unit has two symbols on the housing for correct orientation.

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☞ Find the "backrest" symbol and set the drive unit with its attachment hole over the metal claws on the crossbar [a] of the backrest.

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The metal claws should snap easily but securely into the drive unit.

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☞ Swing the drive unit up until the other attachment hole (with the "thigh rest" symbol) receives the metal claws [c] on the crossbar [b] of the thigh rest.

The metal claws should snap easily but securely into place.

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☞ Hold the drive unit with one hand and push the two cover slides back into place with the other.

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☞ Place the reclining surface frame assembly on the floor to one side with the drive unit facing up.

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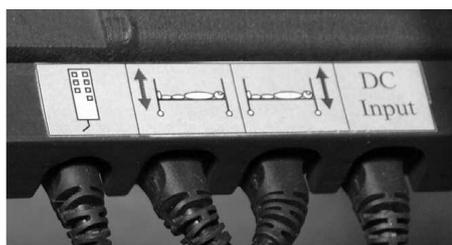
Take special care in assigning the drive motors for the height adjustments. These must not be interchanged. Only one lower foot position should be possible.

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☞ Place the drive motor cables and the hand switch cable under the reclining surface frame.

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☞ Hang the drive unit cable in the corresponding holder on the foot and head boards.

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The connecting cable for the electric motors must be routed when the nursing bed is at the highest horizontal position to avoid straining the cable.

Make sure that no cable is damaged, that there is no loop in the cables and that the cables are not crushed by moving parts.

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When exchanging single electrical components, always make sure that their plugs have been plugged back into the central drive unit until they lock and the plug covering strips have been re-affixed.

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Special care is to be taken when positioning the connecting cable. Make sure that the cable is not damaged and there is no loop in it! When moving the bed around, take care that the castors do not run over the connecting cable! When adjusting the bed the power cable should not get caught between moving parts!

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## LOCATION REQUIREMENTS

■ There must be enough space available to be able to adjust the bed. No furniture, window sills and the like should be in the way.

■ The space under the bed must remain clear.

■ Check before use on parquet flooring, that the castors do not cause discoloration of the varnish. Use on ceramic tiles, carpet, laminate and linoleum is harmless.

BURMEIER is not liable for any damage to floors due to daily operation.

■ A properly installed 230 volt power outlet must be available, near the bed.

■ Take care when attaching further accessory equipment (e.g. compressors for positioning systems, etc.) that their attachment is safe and their functionality is not impaired. Especially important is hereby also the safe placement of all movable connecting cables, tubes, etc. If you have any further questions or need clarification contact the manufacturer of the accessory equipment or BURMEIER.

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Observe the following points when setting up the bed to minimize possible fire hazard.

Instruct the user and the authorized personnel in these points!

- If possible use only fire-retardant mattresses and bed sheets.
- Avoid smoking in bed because of the danger of fire.
- Use only technically faultless appliances (e.g. heating blankets) and other electronic devices (e.g. lamps, radios).
- Take care that these devices are only used for the purpose intended and that they cannot accidentally get on top of or underneath the bed sheets (Fire hazard because of overheating).
- Absolutely avoid the use of extension cords or multi outlet extension cords under the bed (Leaking liquids can cause a fire hazard).
- Connect the bed directly to an electrical outlet. Do not use extension cords and/or multiple outlet extension cords.

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#### REACHING OPERATIONAL READINESS

- Clean and disinfect the bed before first time use and before every further use.

#### DISASSEMBLY

- ☞ Lower the bed into the middle horizontal position.
- ☞ Pull the power plug out of the socket.
- ☞ Remove the security screws in the guide bars.
- ☞ Remove all side rail beams consecutively.
- ☞ Insert the plastic slide back into the guide bar.
- ☞ Loosen the screws and raise both wooden boards from the tubular frames of end boards one by one.
- ☞ Loosen the Allen screws from the reclining surface frame on the foot section, to remove the tubular frame from the foot section; next loosen the Allen screws from the reclining surface frame on the head section, to remove the tubular frame from the head section.

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## HAND SWITCH

The electrical adjustments can be made using the hand switch [8].

The adjustable range of every function is electrically/mechanically possible only within a predetermined range.

Locking mechanisms have been built into the hand switch for safety reasons. For the safety of the user, adjustment functions can be locked at the hand switch, if the condition of the user warrants it.

- ◆ The hand switch, with its elastic hook can be hooked onto the bed.
- ◆ The spiral cable allows for greater mobility.

## OPERATING THE HAND SWITCH

- Train the user in the functions of the hand switch.
- The drive motors will run as long as the button is held down.
- The adjustment functions, except for the lowered foot position, can be used in both directions.
- Basically the functions of the buttons are as shown :
  - ▲ raise
  - ▼ lower

### Adjusting the backrest

Push this button if you want to adjust the backrest angle.

### Adjusting the thigh rest

Push this button to adjust the thigh rest angle.

### Setting up the Auto-contour

Press this button if you want to adjust the angles of the back and thigh rest at the same time. This prevents the user sliding to the foot end.

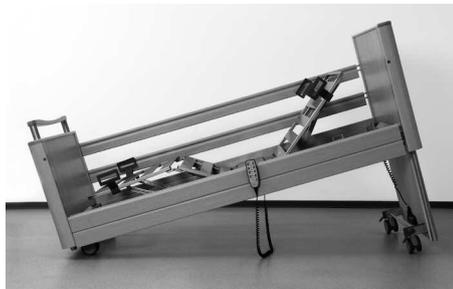
### Adjusting the reclining surface height

Push this button if you want to adjust the height of the reclining surface.

If the bed is in the lowered foot position, it will automatically be adjusted to horizontal level, when it is readjusted to the lowest or highest position.

### Adjusting to the lowered foot position

Push this button if you want to adjust to the lowered foot position. With this button ▼ only the function "Down" is possible.

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■ Unlock the foot brake of both castors on the head or foot section to avoid damaging the floor.

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■ When the bed is reclined a pinch point can occur at the foot end. For this reason the bed must only be lowered after it has been ensured that there is no person situated in the hazard area.

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In case the condition of the user is so critical that any adjustment of the beds by means of hand switch causes danger, the operator has to disable the bed immediately. The nursing bed remains in the position it had at the time of disconnecting.

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Do not turn the locking key with force over the stop of the lock! The lock or the whole hand switch can be damaged.

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The locking key is delivered attached to the bed with a cable tie on the hand switch. The locking key is not intended for use by the user. It must be removed from the hand switch. The operator or a person authorized by the physician should keep the **locking key**.

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■ Turn the respective lock on the hand switch with the locking key clockwise in the locking position.

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The color of the respective display will change from green to yellow.

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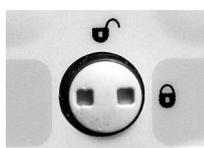
Lock is vertical

Color of the display is: **green**

Buttons can be pushed ("Clicking sound")

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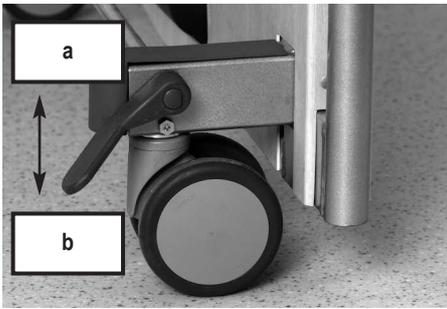
#### Drive disabled

Lock has been turned clockwise by about 90°

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## CASTORS

The bed stands on four steerable, castors [14]. They can be locked in pairs with two foot levers [17].

- ☞ **Braking [b]:** Push the foot lever down.
- ☞ **Driving [a]:** Push the foot lever down in the opposite direction. To safely steer it, push the bed only at the foot end.

- In order to avoid toe injuries, wear closed shoes (and not sandals) when operating the bed.
- Make sure that at least all castors have their brakes applied.

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## SIDE RAILS

Side rails conveniently prevent the user from falling out of bed accidentally. They do not, however, prevent a user from leaving the bed.

### Engaging

- ☞ Pull the side rail beams [6] on one end after the other until they lock in its highest position. You should not be able to move them up or down.
- ☞ Check that the side rail beams are locked into place by pushing on them from above.

### Disengaging

- ☞ Lift the side rail beams slightly.
- ☞ Press the locking button [19] and slowly lower the side rail beams.

### Special safety notes for use of side rail beams:

- Only use safely lockable, technically perfect, undamaged side rail beams with proper gap dimensions
- Make certain that only suitable Original BURMEIER side rail beams are used, which have been approved for use with the bed model you are using or that are already integrated in the bed.
- Before using side rail beams evaluate and take into account the condition and any special problems of the user

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### GENERAL

Cleaning is the most important requirement for a successful chemical disinfection.

Generally a routine cleaning of the bed, if used by the same user, is hygienically sufficient.

Disinfection of the bed frame is only necessary in case of visible contamination with infectious or potentially infectious material (blood, stool, pus) or by presence of an infectious disease under the direction of a physician.

**The bed must be cleaned and washed down with disinfectant in case of a user change!**

**Before beginning the cleaning, please note:**

- Never clean the transformer unit when cleaning the bed. Remove the transformer unit from the socket and avoid contact with water. These components may be cleaned only with a slightly damp cloth.
- If the bed is not connected to a transformer unit, seal the plug coupling on the bed using the attached lid. This prevents faults that could otherwise be caused by moisture.
- Make sure that all the other plugs for the drive components are properly plugged in and secured against being pulled out.
- None of the electrical components must show any visible damage; otherwise water or cleaning solution could penetrate. This could lead to malfunction or damage to the electrical components.
- Before operating the bed again, make sure that there is no residual moisture on the electrical contacts, by drying or blowing on the power plug.
- The electrical components must not be cleaned with a water jet, a high pressure cleaner or other similar device! Clean only with a moist towel.
- If you suspect that water or any other form of moisture has penetrated the electrical components, pull the power plug out of the wall socket immediately. If already disconnected from the power supply, make sure it is not plugged in again. Clearly mark the nursing bed as "Defective" and take it immediately out of service. Report this to the operator promptly.

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■ Failure to follow these directions could lead to major damage of the bed and its electrical components and lead to subsequent malfunctions.

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## CLEANING AND DISINFECTION PLAN

- Remove the bed sheets and give them to the laundry.
- Clean all surfaces including the slatted boards and the reclining surface made of plastic inserts or wire mesh base with a mild and environmentally safe cleaning solution. This also applies to the hand switch.
- If the bed has visible contamination e.g. infectious or potentially infectious material has polluted it, it must be disinfected by wiping after cleaning.
- Disinfecting the castors is only necessary when visibly contaminated by infectious or potentially infectious material.

**Notice:** Continuous disinfection is only necessary for users with a treatment resistant pathogen within the hospital.

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## INSTRUCTING THE USER AND QUALIFIED PERSONNEL

To guarantee the correct procedure for cleaning and disinfection is followed, we recommend to correctly instruct the handler and qualified personnel. Thereby communicating that the following points need to be observed:

- The clean bed is to be transported to the user's home in such a way that it does not become polluted or contaminated.
- When disassembling the bed we recommend cleaning and disinfecting it right away. The qualified personnel should be familiar with the steps necessary to thoroughly clean and disinfect (the operator will provide the individual steps necessary). The disinfectant must be appropriate for the surface being cleaned.

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- The qualified personnel must be equipped with water-tight one-time-use aprons and gloves.
- Only fresh and clean towels must be used to clean and disinfect, these must then be given to the laundry.
- After cleaning the bed, the qualified personnel must also disinfect their hands before going to the next user. An appropriate disinfectant dispenser should be part of the qualified personnel's gear.

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## CLEANING SUPPLIES AND DISINFECTANTS

Make use of these recommendations to keep the bed serviceable for a long time.

- The surfaces must be intact; any damage should be repaired immediately.
- We recommend a (damp) wash down. When choosing the cleaning solution make sure it is mild (gentle to skin and surfaces) and environmentally friendly. In general a regular household cleaner can be used.
- For cleaning the coated metal parts use a wet cloth with regular household cleaners.

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■ Despite the excellent mechanical resistance of the bed, scratches, markings, etc. which penetrate the coating should be resealed using a suitable medium to prevent the penetration of moisture. Contact BURMEIER or your specialized dealer with any questions.

■ Do not use scouring agents, stainless steel cleaners, abrasive cleaning agents or scouring pads. These substances can damage the surface.

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#### **Notice:**

Surface disinfectants based on aldehyde generally have the advantage of working on a large array of pathogens, have a relatively low protein error and are environmentally friendly. Its main disadvantage is the allergy-causing and irritation potential. Compounds on the basis of glucopro-tamine do not have this disadvantage and are very effective, though mostly more expensive. Disinfectants on the basis of chlorine splitting combinations, with prolonged contact or in too high doses, can be corrosive to metals, plastics, rubber and other materials. In addition to this, the compounds have a high protein error, are irritants to the mucous membranes and are not environmentally friendly.

#### **CONTACT WITH DISINFECTANTS**

- Follow the guidelines for each product and manufacturer. Observe the exact dosage! We recommend automatic dosage apparatus.
- Always mix the solution with cold water, to avoid vapors that aggravate the mucous membranes.
- Wear gloves to avoid direct skin contact.
- Do not store the mixed disinfectant in open containers with the cleaning rag inside. Close the container!
- Use closeable bottles with dosage pumps for wetting the cleaning rag.
- After finishing the surface disinfection ventilate the room.
- Use a wipe down disinfecting method, do not use a spray. Using the disinfectant as a spray causes the disinfectant to turn into a mist which could be inhaled.
- Do not use alcohol for larger areas.

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**Notice for Operators**

This bed has been constructed and built to last safely for a long time. With appropriate operation and use, this bed has a life expectancy of 2 to 8 years. The life expectancy is based on the conditions and frequency of use.

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Through repeated transport, assembly and disassembly, improper operation as well as long time use, it is possible that damage, defects and deterioration can occur. These deficiencies can lead to danger if they are not recognized and quickly fixed.

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Because of this there are legal bases for the regular enforcement of tests to ensure the safe condition of this product.

Because of this the following regular inspection and function controls are to be performed by the operator as well as the user.

**Point out these necessary tests to the user.**

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**BY THE USER**

Along with the regular substantial tests by technically qualified personnel, the normal user must also perform a sight and function check at shorter, regular intervals, as well as before a new user.

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- If damage or a malfunction is suspected, take the bed out of service immediately and disconnect it from the power until an exchange or repair can be made!
- Contact your operator regarding exchange or repair of faulty parts.

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#### **Recommendation:**

All electrical and mechanical components should be tested monthly. The connecting cable as well as the hand switch cable should be checked after every mechanical load and after every relocation.

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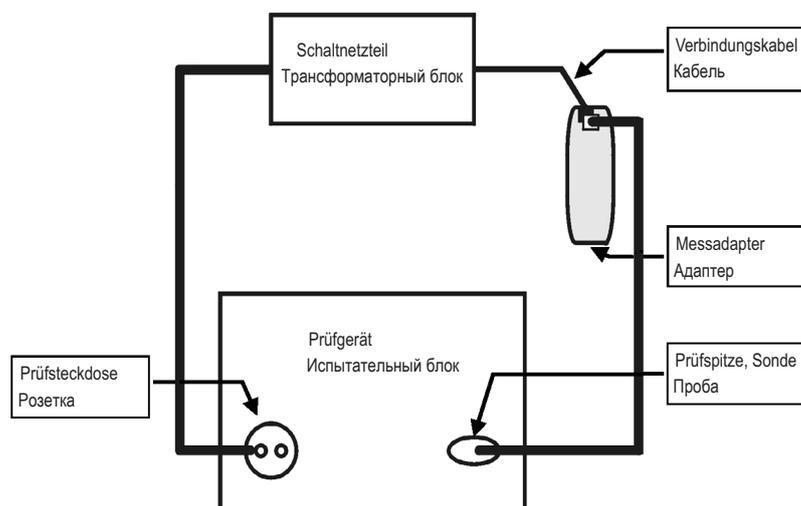
### **BY THE OPERATOR**

- I. Visual inspection
- II. Electric Measurements
- III. Function Test

- ◆The performance of the function test as well as the rating and documentation of the test results can only be performed by experts with the appropriate knowledge, training and experience as well as the tools necessary to perform it according to the rules.
- ◆The assessment and documentation of the test results can only be performed by an expert with the corresponding knowledge, training and experience.
- ◆The testing of the transformer unit only requires measurement of the leakage current. The bed can therefore be used again once the transformer has been replaced with a new unit.
- ◆BURMEIER offers transformer testing as a customer service. Send the transformer unit back to BURMEIER and receive a tested replacement unit. Contact us for further detail.

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**Preparation:**

- Unplug the power plug of the transformer unit from the wall socket.
- Unplug the connecting cable of the transformer unit from the connecting socket.
- Plug the plug of the connecting cable in the adapter (extra accessory; request from BURMEIER).
- Connect the adapter with the test probe of the test unit.
- Plug the transformer power plug into the power socket of the test unit.

**Test method:**

- Leakage current test: direct or differential.
- Leakage current test according to the test unit manufacturers instructions.

**Limit:**

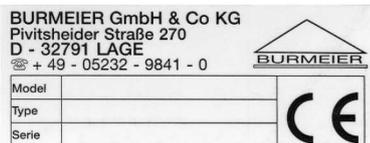
- Leakage current power less than 0.1 mA.

**Test cycle:**

We recommend a yearly visual inspection and performance check. If this check is OK, then it is sufficient to carry out an electrical test every ten years.

If the 2% error quota is shown to have been observed, the test cycle can be extended to a maximum of three years.

If damage or a function error is suspected take the nursing bed out of service immediately and disconnect it from the power until an exchange or repair can be made!

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Model	Bed model
Type	Item Number
Serial	Serial Number

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## SPARE PARTS

The corresponding replacement parts can be obtained from BURMEIER by specifying the item number, the order number and the serial number. The necessary details are found on the type plate under the cross tubing of the reclining surface frame.

## MANUFACTURER'S ADDRESS

In order to maintain operational safety and the right to claim under warranty, only original BURMEIER replacement parts may be used!

For ordering replacement parts, customer service requests and other questions, please contact:

**Burmeier GmbH & Co. KG**  
(A subsidiary of the Stieglmeyer Group)

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## REPLACEMENT OF ELECTRICAL COMPONENTS

### Dangerous voltage, electric shock possible

- Before doing any work unplug the main power cable from the wall socket!
- Any work on the electrical equipment may only be performed by customer service, the motor manufacturer or qualified and authorized electricians under consideration of the safety guidelines!

- Only dismantle the drive motors when the reclining surface is in the horizontal position, otherwise there is the risk of being injured by falling parts of the mattress base.
- All the drive components are maintenance-free and must not be opened. In the event of a malfunction, the corresponding components should always be replaced in full!

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### Exchange of the Hand switch

- ☞ Unplug the power plug from the wall socket.
  - ☞ Turn the plug cover on the drive unit to remove it.
  - ☞ Unplug the plug at the end of the spiral cable of the hand switch from the connector of the drive unit.
  - ☞ Lay the cable of the new hand switch so that it cannot be damaged by moving parts of the bed.
  - ☞ Insert the plug at the end of the spiral cable of the new hand switch into the connector of the drive unit.
- Take care that the O-ring on the plug is not damaged, otherwise the connection may not be moisture-proof.
- ☞ Carry out a function test of the electrical adjustments after the exchange.

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## MEASUREMENTS AND DIMENSIONS

## Assembled bed with side rails:

- Reclining surface
- External dimensions
- Total weight
- Safe load

## Disassembled bed:

- Reclining surface excluding motor
- Undercarriage, including motor
- Head board encasement
- Foot board encasement without strip handle
- Strip handle
- 4 side rail bars
- Trapeze pole
- Two-side drive

## SPECIAL FEATURES

- ◆ Electrical height adjustment of the reclining surface from about 22 cm to 77 cm.
- ◆ Electrical adjustment of the backrest from 0° to about 70°.
- ◆ Electrical adjustment of the leg rest from 0° to about 35°.
- ◆ Electrical inclination of the reclining surface to the foot end from about 10°.
- ◆ Simultaneous adjustment of the back and thigh rest (Auto-contour).
- ◆ Moves on four castors. Each axle can be locked separately.

## INOVIA II

- ◆ Reclining surface of 200 x 90 cm, divided into four areas; Outside dimensions are about 219 x 105 cm.
- ◆ Side rails on both sides can be lowered.

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**Motors for reclining surface height adjustment**

Type  
 Power / Size / Lift  
 Input Voltage  
 Operating Time  
 Protection

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**Drive unit for backrest and thigh rest**

Type  
 Power (dynamic); backrest / thigh rest  
 Input Voltage  
 Output Volts  
 Operating Time  
 Protection

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**Explanation of Icons:**

◆ Unit with Type B

(Special protection against electric shock)

◆ Protection Class II device, double insulation

◆ Transformer with thermal fuse block

◆ Safety transformer

◆ Caution! Consult instruction manual

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♦ The electrical equipment offers protection from internal dust deposits and is splash-proof from all sides

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♦ The electrical equipment is dust-proof from internal dust deposits and is splash-proof from all sides

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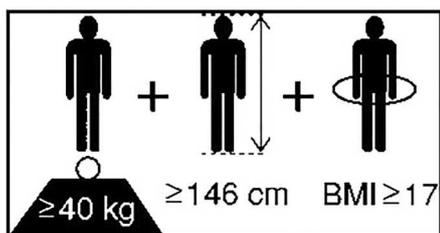
♦ Minimum user sizes/weight:  
Height: 146 cm,  
Weight: 40 kg,  
Body Mass Index „BMI“: 17

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#### Noise level during motor adjustments

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The ambient conditions stated below must be observed:

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### DISPOSAL INSTRUCTIONS

- ♦ Exchanged electronic components (drives, control unit, hand switch, etc.) of these nursing beds are considered electronic scrap.
- ♦ No longer useable rechargeable batteries have to be disposed of according to regulations and do not belong in the house garbage.
- ♦ With all disposed components the owner must make sure that they are not infectious or contaminated.
- ♦ In the event that the nursing bed must be scrapped, the plastic and metal parts used must be disposed of separately, following the proper disposal procedure.
- ♦ Pneumatic springs are under high pressure!
- ♦ Hydraulic units are made up of mostly metal and plastic. Before disposing according to manufacturer guidelines the oil has to be drained and disposed of properly.
- ♦ Contact your local community, disposal facility or our service department with any questions.